


Personal Protective Equipment
(PPE) for Healthcare Personnel

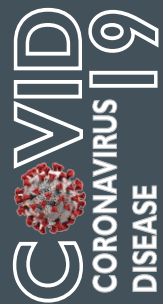


Face Shield

Safety Protection Comfort



COMPATIBLE WITH PPE's

Designed to work with other Personal Protective Equipment (PPE's) and prescription glasses

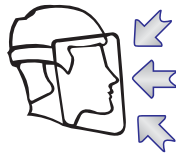

COMPLETE PROTECTION

Completely covers sides & length of face. Protects face, eyes, nose, mouth and surrounding areas




COMFORTABLE & EASY TO USE

Light weight and ergonomic design. Foam lining offers comfort fit and better airflow



ANTI FOG

Anti-fog clear plastic material provides good visibility and clarity

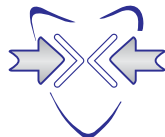


ADJUSTABLE

Shield & Holder adjustable for optimum fit for all head sizes



REUSABLE FRAME

Superbly designed reusable frame and quick change disposable Visor

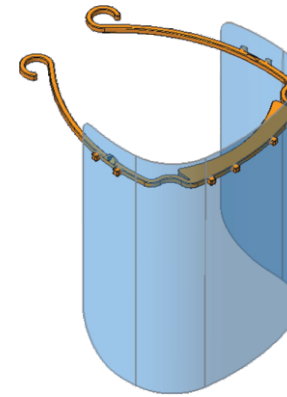

ALL ROUND PROTECTION

FOREHEAD
EYES
NOSE
MOUTH



Instructions for use

- Carefully remove protective films from both faces of the transparent Shield
- Assemble transparent Shield with plastic Holder by press fitting on the centre tap first
- Gently press fit the Shield into left and right hand tabs
- Wear the Shield and adjust the Holder as per proper fitting

Instructions for replacement

- Transparent Shield only, available on order
- Remove transparent Shield from Holder tabs one by one starting from any of the side faces
- Assemble new transparent Shield by press fitting as per usage guidelines
- It is advisable to replace the whole set after a week's worth of use for optimum quality

Presentation

50 INDIVIDUAL SET 100 BULK SET 200 BULK SET
PCS Individually packed PCS PCS

MATERIAL : PC ABS

POSTAL ADDRESS:
INJECTOPLAST PVT. LTD.
205- Leela Palace,
7/85, Tilak Nagar
Kanpur - 208002, India.

MANUFACTURED BY:
INJECTOPLAST PVT. LTD.
485/B-486 Amiliha Chaubepur,
NH 91, G.T. Road,
Kanpur 209203 (U.P)

CONTACT :

ROHIT SHAH / ABHISHEK SHAH
+91 9839104777, +91 512 2334500

injecto@injectoplast.com

www.injectoplast.com